



! *Ensure you have an empty DVD-R available*

Switch the console on and start the physiology section

1. In the **Patient** tab select **Retrieve**

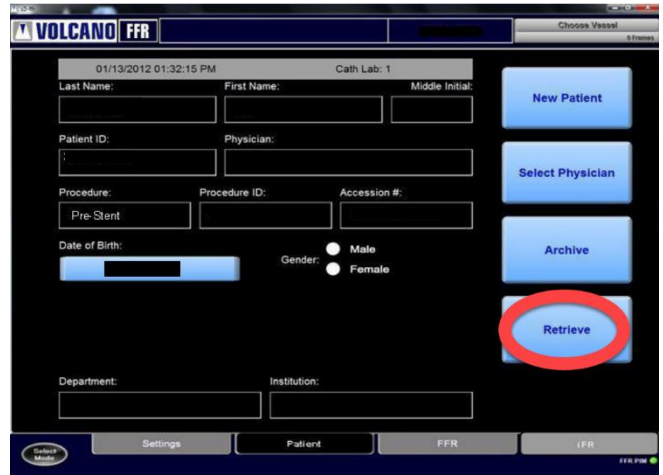


2. Select the case needed and click on **Retrieve**

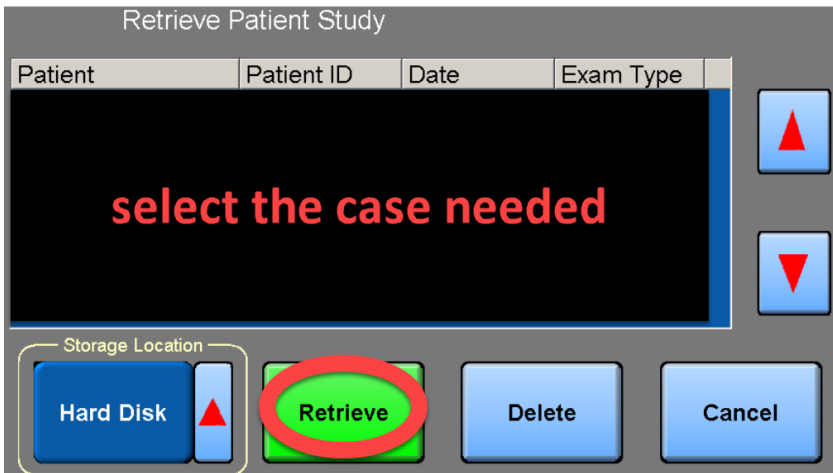

The case with the pressure tracing are now visual

3. Click on the **Patient** tab and select **Archive**

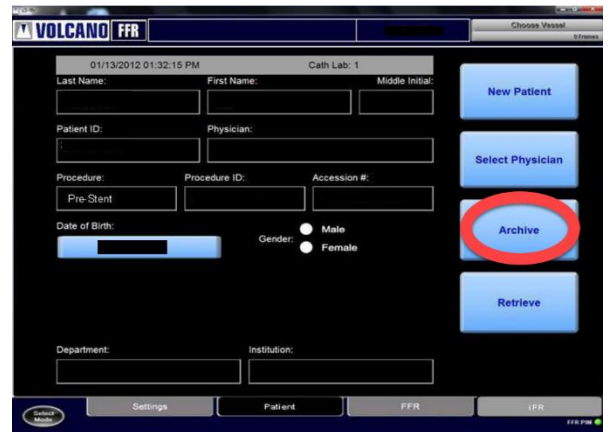

4. Select in the storage section **DVD**

Check the box **Archive the study anonymously**

Load a **DVD-R** and click on **Save**




5. **Anonymised Screenshots** can be taken from **DVD** for upload in **Synchrony EDC**

How to Save iFR data (screenshots) for iMODERN on USB stick (back-up solution)


! *iFR Images archived on an USB stick are not anonymised, this should then be done manually prior to uploading the Images to the Sponsor.*

Place the memory stick in the USB port and Start the console

1. In the **Patient** tab select **Retrieve**

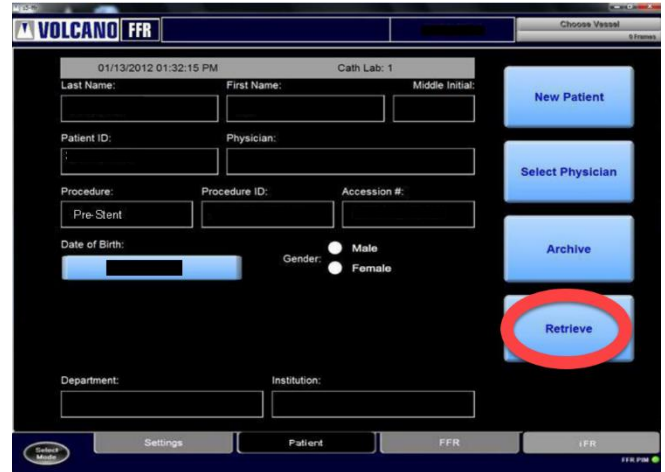


2. Select the case needed and click on **Retrieve**

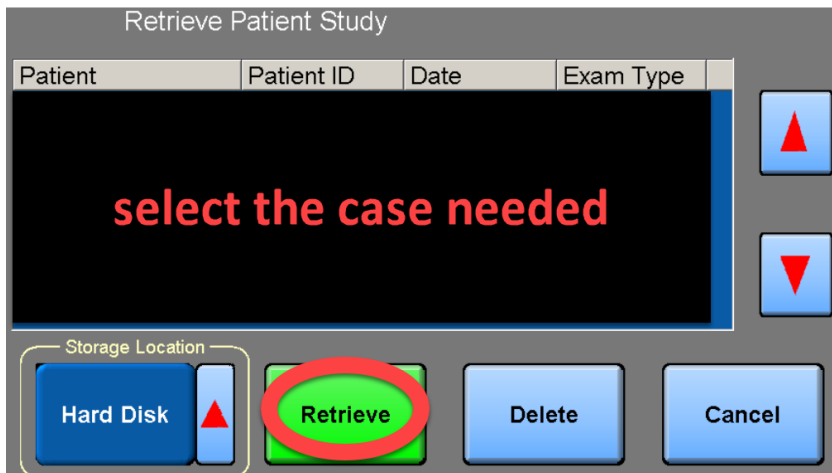


The case with the pressure tracing are now visual

3. Click on the **Patient** tab and select **Archive**

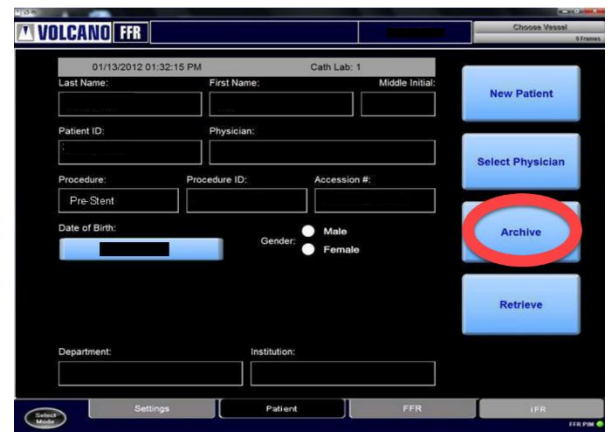

4. Select in the storage section **Memory stick**

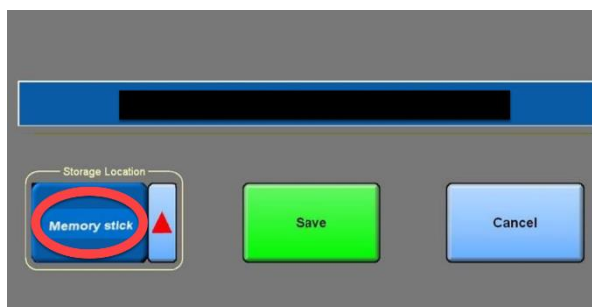

5. **NON Anonymised Screenshots** can be taken from **USB stick** but should be **manually anonymised prior to upload in Synchrony EDC**